
## Revisions

SO 0191-5

Rev from	Rev to	ECO #
1206	2010/06	5347-10

**Notes:**

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1. BD Cat. Number 261197, 261198, 261207
2. Blank (Sheet) Size: Length: 11"      Width: 27"  
 Number of Pages: 12      Number of Sheets: 1  
 Page Size: Length 11"      Width 4 1/2"      Final Folded Size: 4 1/2" x 1 7/8"
3. Style (see illustrations below): #4

4. See Specification Control Number D11835RM for Material Information
5. Ink Colors: Printed two sides  Yes       No  
 No. of Colors: 1      PMS# 2755 (blue)
6. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	<b>Becton, Dickinson and Company</b> 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: L001190		Category and Description Package Insert, Nitrate A, B & C Reagent Droppers	Sheet: 1 of 13 <hr/> Scale: N/A	A

# BD Nitrate A, Nitrate B and Nitrate C Reagent Droppers

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## INTENDED USE

**BBL™ Nitrate Reagent A, BBL Nitrate Reagent B and BBL Nitrate Reagent C Droppers** are for use in determining the ability of bacteria to reduce nitrate to nitrite or free nitrogen gas.

## SUMMARY AND EXPLANATION

The *Enterobacteriaceae*, many other gram-negative bacteria, mycobacteria and fungi reduce nitrate to nitrite. The nitrate-reducing characteristic of a species is constant for certain genera and species. The Nitrate A and B Reagents, when added in equal parts, indicate the presence of a catabolic end product or the absence of nitrate in the medium.<sup>1,2</sup> In the event of a negative reaction, the addition of Nitrate C Reagent shows the presence of nitrate in the medium and confirms the negative reaction.

## PRINCIPLES OF THE PROCEDURE

The reaction indicates the ability of the organism to reduce nitrate (NO<sub>3</sub>) to nitrite (NO<sub>2</sub>), nitrous oxide (N<sub>2</sub>O), nitric oxide (NO) or ammonia (NH<sub>3</sub>). In the presence of any of these end products, the Nitrate A and B Reagents, added in equal proportions, will turn the medium pink or red if the organism is nitrate positive. The formation of other end products that do not turn the medium pink or red is also possible; therefore, the addition of Nitrate C Reagent (zinc dust) is used to detect unreduced nitrate. In the presence of unreduced nitrate, zinc dust will turn the medium pink or red. The absence of color after the addition of zinc dust indicates that nitrate has been reduced to nitrogen gas (N<sub>2</sub>) and the organism is nitrate positive.

## REAGENTS

Nitrate A Reagent Droppers contain 0.5 mL of 0.8% Sulfanilic Acid in 5N Acetic Acid. Nitrate B Reagent Droppers contain 0.5 mL of 0.6% N,N-Dimethyl-alpha-naphthylamine in 5N Acetic Acid. Nitrate C Reagent Droppers contain 1 g of zinc dust.

## Warnings and Precautions:

For *in vitro* Diagnostic Use.

Follow proper laboratory procedures in handling and disposing of infectious materials.

Nitrate A Reagent causes burns. Keep locked up and out of reach of children. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable protective clothing, gloves and eye/face protection.

Nitrate B Reagent causes burns. When using, do not eat or drink. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable protective clothing and eye/face protection. Nitrate B Reagent, although not classified as carcinogenic by the U.S. Occupational Safety and Health Administration (OSHA), is closely related to alpha-naphthylamine, which is considered a carcinogen.<sup>3</sup>

Nitrate C Reagent in contact with water liberates extremely flammable gases. Spontaneously flammable in air. Keep out of reach of children. Keep container tightly closed and dry. In case of fire, use sand, carbon dioxide or powdered extinguishing agent. Never use water.

**Storage Instructions:** Store Reagent Droppers at room temperature 15 – 30°C (59 – 86°F). Protect from light. Each dropper is good for one day's use after breaking ampule.

**Product Deterioration:** These reagents are hermetically sealed in ampules, which afford protection from chemical instability until the expiration date. A change in color of reagents A and B from clear or light yellow to brown indicates improper storage, which may cause weaker reactions.

## PROCEDURE

**Materials Provided:** Nitrate A Reagent Droppers, Nitrate B Reagent Droppers or Nitrate C Reagent Droppers, as ordered.

**Materials Required But Not Provided:** Ancillary culture media, reagents, quality control organisms and laboratory equipment as required for this procedure.

### Test Procedures:

To use Nitrate A or B Reagent Droppers, hold upright and **POINT TIP AWAY FROM YOURSELF**. Grasp the middle with thumb and forefinger and squeeze gently to break ampule inside the dropper. **Caution: Break ampule close to its center one time only. Do not manipulate the dropper any further as the plastic may puncture and injury may occur.** Tap bottom of dropper on the tabletop a few times. When ready for use, invert for convenient drop-by-drop dispensing of reagent. To use Nitrate C Reagent Droppers, remove the cap and invert to dispense the zinc dust.

**Liquid Medium Method:** Method for *Enterobacteriaceae* and nonfermenting gram-negative rods.

1. Inoculate Nitrate Broth with Durham tube with the organism to be tested using an 18 – 24 h culture.
2. Incubate inoculated tubes at 35 – 37°C for 18 – 24 h. Occasionally, prolonged incubation is needed to achieve a positive reaction for some organisms.<sup>1</sup>
3. Remove from incubator and examine Durham tube for presence of gas. If gas is present, the organism is nitrate positive. If no gas is present, continue with step 4.
4. Add 10 drops of Nitrate A and 10 drops of Nitrate B to tube. (Alternative procedure: Remove approximately 2 mL of broth and place in clean 12 x 75 mm test tube. Add 5 drops

of Nitrate A and 5 drops of Nitrate B to tube. Shake gently to mix reagents. If negative, remaining broth may be reincubated and tested the next day.)

5. Observe broth for pink/red color development within 2 min.
6. If pink/red color develops, the organism is nitrate positive.
7. If no color develops within 2 min, add a small amount of Nitrate C (one inversion) to the tube.
8. If the medium turns pink/red within 2 min, nitrate is present (i.e., not reduced by the organism) and the organism is nitrate negative.
9. If there is no color change noted within 2 min, nitrate is not present in the broth (i.e., the organism reduced the nitrate to nitrogen gas) and the organism is nitrate positive.

**Semisolid Medium Method:** Method for fastidious gram-negative rods, anaerobes and gram-positive organisms.

1. Inoculate Indole Nitrite Medium with the organism to be tested using an 18 – 24 h culture.
2. Incubate inoculated tubes at 35 – 37°C for 18 – 24 h. Occasionally, prolonged incubation is needed to achieve a positive reaction for some organisms.<sup>1</sup>
3. Add 10 drops of Nitrate A and 10 drops of Nitrate B to tube. (Alternative procedure: Remove approximately 2 – 3 mL of medium and place in clean 12 x 75 mm test tube. Add 5 drops of Nitrate A and 5 drops of Nitrate B to tube. Shake gently to mix reagents. If negative, remaining broth may be reincubated and tested the next day.)
4. Observe broth for pink/red color development within 2 min.
5. If pink/red color develops, the organism is nitrate positive.
6. If no color develops within 2 min, add a small amount of Nitrate C (one inversion) to the tube.
7. If the medium turns pink/red within 2 min, nitrate is present (i.e., not reduced by the organism) and the organism is nitrate negative.
8. If there is no color change within 2 min, nitrate is not present in the medium (i.e., the organism reduced the nitrate to nitrogen gas) and the organism is nitrate positive.

**Other Methods:** These reagents can be used in any variation of the nitrate reduction test that requires Nitrate A, Nitrate B and Nitrate C.

**User Quality Control:** Examine the reagents for signs of deterioration (see "Product Deterioration"). *Escherichia coli* ATCC™ 25922 will yield a positive result, while *Acinetobacter baumannii* ATCC 17961 will yield a negative result.

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent CLSI (formerly NCCLS) guidance and CLIA regulations for appropriate Quality Control practices.

## RESULTS

Nitrate Positive:

- a. Gas in Durham tube *or*,
- b. After addition of reagents, pink/red color develops *or*,
- c. After addition of reagents no color develops *and* after addition of zinc dust no color develops.

Nitrate Negative: After addition of reagents no color develops *and* after addition of zinc dust pink/red color develops.

## LIMITATIONS OF THE PROCEDURE

Allow at least 2 min for the color to develop before considering the nitrate test negative.

Nitrate C (zinc dust) must be added to any negative reaction to eliminate the possibility of a false negative reaction due to the organism's ability to reduce the nitrate to nitrogen gas.

Use only pure cultures. Mixed cultures may produce misleading results.

Use only the recommended amount of zinc dust. Excess zinc dust may cause false reactions.<sup>4</sup>

## PERFORMANCE CHARACTERISTICS

An internal study compared BBL Nitrate A, B and C Reagents to another set of commercially available Nitrate Reagents.<sup>5</sup> As noted in the table, all reactions were identified for the 7 ATCC strains tested using both sets of Nitrate Reagents.

Organism	Expected Results	BBL Reagents	Commercial Reagents
<i>Acinetobacter anitratus</i> ATCC 7961	–	–	–
<i>Escherichia coli</i> ATCC 25922	+	+	+
<i>Pasteurella multocida</i> ATCC 11096	+	W+	W+
<i>Branhamella cattharalis</i> ATCC 38098	+	W+	W+
<i>Eikenella corrodens</i> ATCC 7622	+	+	+
<i>Branhamella cattharalis</i> ATCC 25238	+	W+	W+
<i>Enterobacter cloacae</i> ATCC 23355	+	+	+

Key	
–	Negative
+	Positive
W+	Weak Positive

## AVAILABILITY

Cat. No.	Description
261197	<b>BBL™</b> Nitrate A Reagent Droppers, 50 x 0.5 mL.
261198	<b>BBL™</b> Nitrate B Reagent Droppers, 50 x 0.5 mL.
261207	<b>BBL™</b> Nitrate C Reagent Droppers, 50 x 1 g.

## REFERENCES

1. MacFaddin, J.F. 1980. Biochemical tests for the identification of medical bacteria, 2nd ed. Williams & Wilkins, Baltimore.
2. Forbes, B.A., D.F. Sahm, and A.S. Weissfeld. 1998. Bailey and Scott's diagnostic microbiology, 10th ed. Mosby, Inc., St Louis.
3. Balows, A., W.J. Hausler, Jr., K.L. Herrmann, H.D. Isenberg, and H.J. Shadomy (ed.) 1991. Manual of clinical microbiology, 5th ed. American Society for Microbiology, Washington, D.C.
4. Porres, J.M., and V. Porter. 1974. Rapid nitrate reduction test. Am. J. Med. Technol. 40: 257-259.
5. Data on file, BD Diagnostic Systems.

# BD Nitrate A, Nitrate B and Nitrate C Reagent Droppers

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Benex Limited