


BD GeneOhm™ StrepB

Vaginal/Rectal Specimen Collection Protocol


one

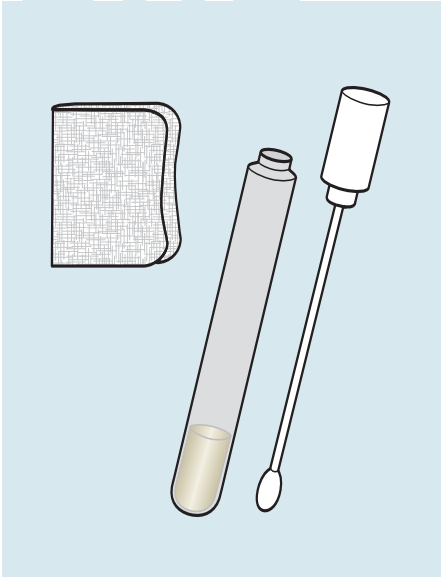

Before collecting the specimen, **WIPE AWAY** excessive amount of secretion or discharge from the vaginal area.


Use the recommended swab to collect the vaginal/rectal specimen.

two

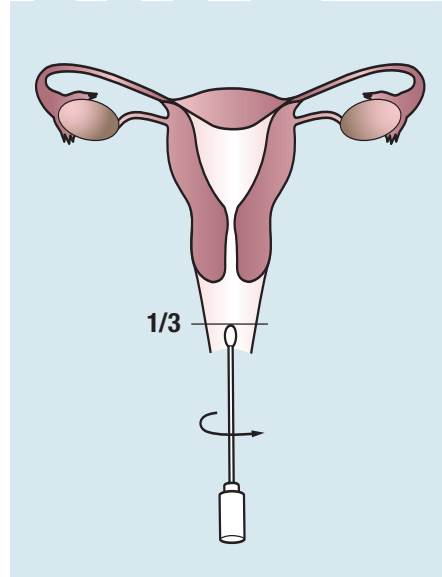


Insert the swab into lower one-third of the vagina and sample secretions from mucosa.

three

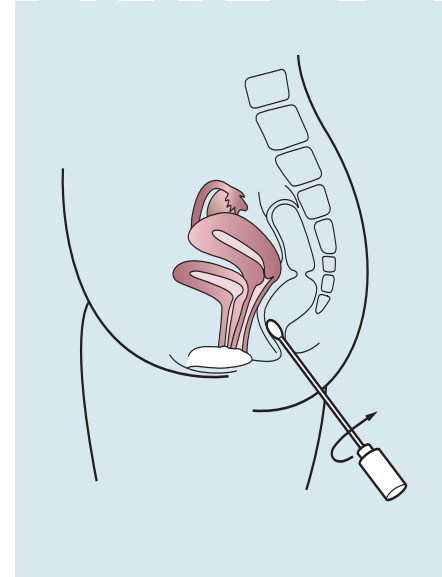

Carefully insert the same swab approximately 2.5 cm (1 inch) beyond the anal sphincter and gently rotate to sample anal crypts.

four

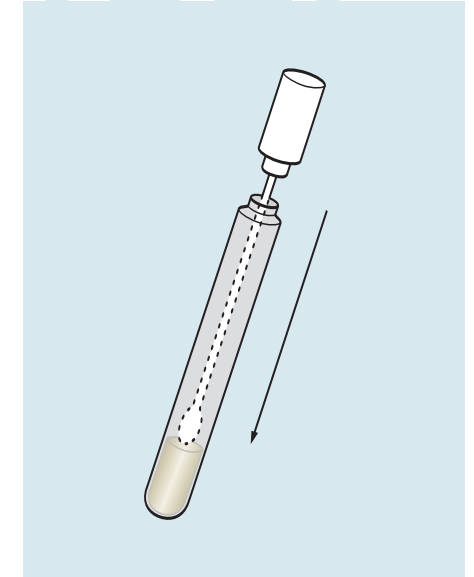

Return swab to its container and send to the laboratory immediately.

Not for Lab Use • See package insert for test protocol.

BD, BD logo and all other trademarks are property of Becton Dickinson and Company.
Copyright © 2006 BD
Patent protection pending



BD Diagnostics
6146 Nancy Ridge Drive
San Diego, California 92121
www.geneohm.com
888-GENE OHM

BD Canada
2771 Bristol Circle
Oakville, ON
L6H 6R5
800-268-5430