



## BD SoloShot™ LX

BCG Auto-Disable Syringe

The BD SoloShot™ LX Syringe is designed to prevent the syringe from being reused.

Unlike an ordinary syringe, **the plunger can only be pulled back once.**

Before using the syringe with patients, practice several times with normal saline or sterile water to become familiar with the unique features.

Everyone deserves the chance to grow up.

BD – Delivering the vaccine promise.



[www.bd.com/immunization](http://www.bd.com/immunization)



1 Becton Drive, MC 204  
Franklin Lakes, NJ 07417  
tel: 201.847.5891  
fax: 201.847.4845  
30 Tuas Avenue 2  
Singapore 639461  
tel: 65.6861.0633  
fax: 65.6860.1593  
Denderstraat 24 B-9320  
Erembodegem Aalst  
Belgium  
tel: 32.53.720211  
fax: 32.53.720200

Unit 401-403  
318 Pidemco Tower  
318 Fuzhou Road  
Shanghai  
People's Republic of China  
tel: 86.21.6391.2678  
fax: 86.21.6391.2698  
Carretera General San Martin  
16500  
Sitio 33, Colina  
Santiago, Chile  
tel: 562.460.0380  
fax: 562.460.0306  
Signature Towers B, 6th Floor  
South City I  
Gurgaon 122001  
Haryana, India  
tel: 91.124.638.3566  
fax: 91.124.638.3224

BD, BD Logo and SoloShot are trademarks of Becton, Dickinson and Company. © 2003 BD BD 04/03 7015-E.

## BD SoloShot™ LX

BCG Auto-Disable Syringe

### *Instructions for Use*



Helping all people live healthy lives



# BD SoloShot™ LX

BCG Auto-Disable Syringe

## Instructions for Use

1 →



Check that the syringe package is undamaged and unopened. Discard if damaged or opened.

2 →



Peel open the end of the package and remove the syringe.

Remove the shield from the needle. **Do not move the plunger, as this will disable the syringe.**

4 →



To remove air bubbles, remove the needle from the ampoule or vial. Holding the syringe upright, tap the barrel to bring the bubbles toward the tip of the syringe. Then carefully push the plunger back to the dose mark.

3 →



Insert the needle into open ampoule or inverted vial.

**Keep the needle tip in the fluid at all times. Do not draw air into the syringe. It may lead to an inaccurate dose.** Pull the plunger back slowly to fill the syringe. The plunger will automatically stop just past the dose mark.

5 →



Administer the injection according to clinical guidelines.

Depress the plunger completely to deliver the dose. The plunger will automatically lock to prevent the syringe from being reused.

6 →



Place the BD SoloShot™ LX syringe immediately in a needle disposal container. **Do not reshield.**

