

BD Uniject™

Prefill Injection Device

Features:

- All-in-one prefilled injection device
- Single-use, non-reusable drug delivery system
- Compact size
- Tamper-evident packaging
- Full needle isolation

Benefits:

- Simplifies logistics for immunization campaigns in remote areas
- Minimizes risk of disease transmission
- Fast and easy injections
- Economical transport, storage and disposal
- Preferred by health care providers and patients in field trials*

* Data on file at BD Pharmaceutical Systems.



How To Use:

- Hold the BD Uniject™ by the port
- Activate the device by pushing the needle shield into the port until you feel the click
- Remove needle shield
- While still holding the device by the port, inject the patient
- Squeeze the reservoir firmly to inject the medication until the reservoir collapses
- Dispose of properly

A unique way to make immunization safe



Prefilled Drug Reservoir

delivers precise amount of vaccine to reduce waste and cost. Available in 0.5ml and 1.0ml.

Disc Valve

between the reservoir and isolation barrier prevents refilling while allowing for an aspiration test

Port

covers the needle hub, isolates the vaccine prior to use, and connects the needle to the reservoir

Needle Assembly

consists of needle and hub, and is covered by the needle shield until you are ready to inject

World's Sharpest Needle

for greater patient comfort. Double-pointed needle allows you to pierce the reservoir when activating the device.

Tamper Evident Packaging

ensures safe and sterile injection

Compact Size

allows for easy storage and disposal

Innovative, Award-Winning Design

facilitates fast and easy single injections



The BD Uniject™ prefill injection device is a cost-effective way to deliver vaccines and other drugs safely to people in remote or hard-to-reach areas. The BD Uniject™ offers the latest technology to eliminate the need for filling syringes from vials, preventing needle re-use and maintaining the vaccine cold chain up to the point of use.

The BD Uniject™ is a single-use system that delivers the precise amount of medication with a squeeze of the drug reservoir. Once the injection has been given, the device cannot be re-used because the sealed drug reservoir membrane has been broken and a disc valve inhibits refilling.

The innovative, award winning design allows for fast and easy injections, while the compact size allows for easy transport, storage and disposal.

Tamper-evident packaging ensures safe, sterile injections. In addition, the needle is fully isolated to reduce the risk of contamination or accidental drug loss.

And, most importantly, the BD Uniject™ is preferred by patients and healthcare providers alike and has been confirmed in field trials. In addition, leading global health organizations and ministries of health report that BD Uniject™ is easy to use*.*

* Data on file at BD Pharmaceutical Systems.