



# Home Care Patient Guide



Scrub the hub

## Disinfect


- Clean the silicone surface with an alcohol pad or the antiseptic suggested by your health-care provider.
- Clean prior to each access.



Connect securely

## Access

- Using a straight-on approach, attach the syringe or administration set by pushing the tip through the silicone surface and twist to ensure a secure connection.

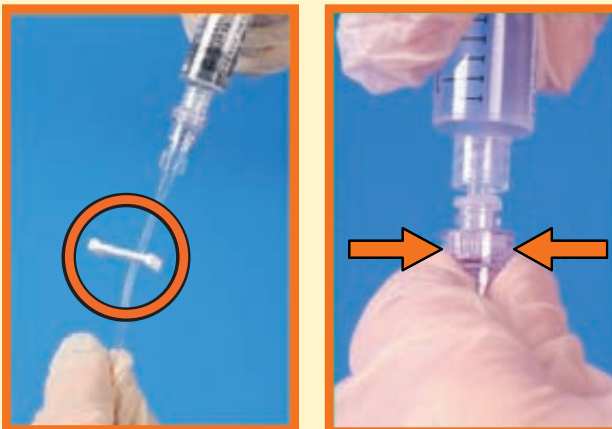

Clamp before disconnect  
Hold the BD Q-Syte device

## Flush

- Flush with the type and volume of solution as directed by your healthcare provider.
- Clamp the extension set or catheter prior to disconnection.
- When disconnecting, hold the BD Q-Syte device firmly in place to prevent accidental removal.



Helping all people  
live healthy lives

NOTE: Refer to package insert for precautions and complete instructions for use.

BD Medical, Sandy, Utah 84070 USA. BD, BD Logo and BD Q-Syte are trademarks of Becton, Dickinson and Company. ©2007 BD

D15130-3A (4/07)