


BD NEXIVA™ CLOSED IV CATHETER SYSTEM

>>>

A NEW STANDARD IN IV THERAPY.

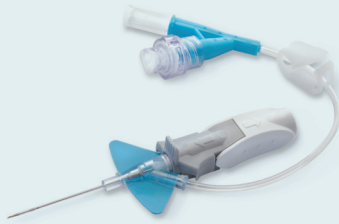

TIPS FOR SUCCESS

SHARP INSERTION:

- > Keep the stabilization platform and finger grips pushed together.
- > DO NOT insert holding only onto the stabilization platform as this will push the needle back into the catheter.
- > Avoid holding in a manner that pushes against the stabilization platform.

EASY VESSEL ACCESS:

- > Look for initial blood return in the catheter near the insertion site.
- > Lower and slightly advance the entire system to ensure the catheter tip is within the vessel.
- > Look for continuous blood flow along the extension tubing to confirm you are still in the vessel.

SMOOTH CATHETER ADVANCEMENT:

- > Push the catheter forward off the needle rather than pulling the needle back.
- > Use the pad of your index finger and not your fingernail behind the push tab.
- > Push down and forward against the push tab.

SIMPLE NEEDLE SHIELD REMOVAL:

- > Pushing down gently on the stabilization platform, pull back on the finger grips until the needle shield releases.
- > Do not touch the push tab during needle removal.

BLOOD CONTAINMENT:

- > Clamp the extension tubing before replacing the vent plug.
- > Secure the vent plug and the BD Q-Syte™ Luer Access Split Septum device before insertion.

EASY SYSTEM ACCESS:

- > Insert the male luer into the BD Q-Syte device using a straight-on approach.
- > Clamp BEFORE disconnecting the flush syringe.
- > Hold bottom of BD Q-Syte device when removing the syringe or tubing during disconnection.
- > Use a straight-off approach when removing the male luer.



BD Medical

9450 South State Street
Sandy, Utah 84070
1.888.237.2762
www.bd.com/infusion