

BD offers **one safety technology** for both your blood collection and injection needs

## BD Eclipse™ Needles

### Blood Collection Needle



For routine blood draws, the BD Eclipse™ Blood Collection Needle, used in conjunction with the BD Vacutainer™ One Use Holder, is the product of choice.

### Hypodermic Needle



The BD Eclipse™ Hypodermic Needle allows the same safety-engineered technology to be used by practitioners when a procedure calls for a hypodermic needle with a syringe to draw blood.

## ORDER INFORMATION

### BD Vacutainer™ One Use Holder

Ref. #	Description	Packaging
364815	One Use Holder	250/Bag 1000/Case

### BD Eclipse™ Blood Collection Needle

Ref. #	Description	Packaging
368607	21g x 1-1/4 in	48/Box 480/Case
368608	22g x 1-1/4 in	48/Box 480/Case

### BD Safety-Lok™ Blood Collection Set

Ref. #	Description	Packaging
367281	with Luer, 21g 3/4 in -12 in Tubing	50/Box 200/Case
367283	with Luer, 23g 3/4 in -12 in Tubing	50/Box 200/Case
367285	with Luer, 25g 3/4 in -12 in Tubing	50/Box 200/Case
367287	with Luer, 21g 3/4 in -7 in Tubing	50/Box 200/Case
367292	with Luer, 23g 3/4 in -7 in Tubing	50/Box 200/Case
367294	with Luer, 25g 3/4 in -7 in Tubing	50/Box 200/Case
367296	without Luer, 21g 3/4 in -12 in Tubing	50/Box 200/Case
367297	without Luer, 23g 3/4 in -12 in Tubing	50/Box 200/Case
367298	without Luer, 25g 3/4 in -12 in Tubing	50/Box 200/Case

### BD Vacutainer™ Luer Adapter

Ref. #	Description	Packaging
367290	Multiple Sample Luer Adapter	100/Box 1000/Case

### BD Eclipse™ Hypodermic Needle

Ref. #	Description	Packaging
305762	23g x 1 in	100/Box 1200/Case
305761	25g x 1 in	100/Box 1200/Case
305764	21g x 1 in	100/Box 1200/Case

Other BD Eclipse hypodermic needle sizes from 18g-27g in lengths from 1/2" to 1 1/2" are also available in addition to those shown above. Visit [www.bd.com/safety](http://www.bd.com/safety) for more information.

For more information on BD Safety-Engineered products, call:

**BD Vacutainer Technical Services:**  
**1.800.631.0174**

**BD Customer Service:**  
**1.888.237.2762**

[www.bd.com/vacutainer](http://www.bd.com/vacutainer)

Trust BD Vacutainer™  
Family of products you can rely on



BD Vacutainer Systems  
Preanalytical Solutions  
1 Becton Drive  
Franklin Lakes, NJ 07417



Helping all people  
live healthy lives

# BD Eclipse™ Blood Collection Needle

# BD Vacutainer™ One Use Holder

Part of the BD Vacutainer™ Venous  
Blood Collection System



# BD Eclipse™ Blood Collection Needle

Offers a simple, effective way to collect blood, while reducing the possibility of accidental needlestick injuries



- **INTUITIVE** and **EASY TO USE**, so there is minimal change to your technique
- **NEEDLE "BEVEL UP"** orientation toward the pink safety shield
- **SAFETY SHIELD** is an **"INTEGRAL"** part of the needle; eliminates the need for secondary safety accessory device
- Immediate **ONE-HANDED** safety feature **ACTIVATION**—no need to activate against a hard surface
- **SAFETY SHIELD** design allows **HANDS** to **REMAIN BEHIND** the **NEEDLE** at all times
- Uses BD PrecisionGlide™ needle technology to ensure **HIGHEST QUALITY** and **NEEDLE SHARPNESS**
- **BD Eclipse™ Hypodermic Needle** offers **COMPLEMENTARY TECHNOLOGY** when hypodermic needles and syringes are used to collect blood samples

## BD Vacutainer™ One Use Holder

Compatible with the entire BD Vacutainer Venous Blood Collection System



BD Eclipse™ Blood Collection Needle



BD Safety-Lok™ Blood Collection Set



BD Vacutainer™ Luer Adapter

### Low-Cost Alternative

- Provides customers who choose to institute a single use holder policy with **ONE PRODUCT** for all of their venous blood collection needs

### Clearly Marked

- Each holder is engraved with the words **"DO NOT REUSE"** and **"SINGLE USE ONLY"** to serve as a reminder for single use holder compliance

### IMPORTANT ONE USE HOLDER SAFETY TIPS

- **DO NOT REUSE HOLDER**
- Holder threads are designed to be used **ONLY ONCE**
- Reuse of holder is potentially unsafe and could result in serious injury or exposure to bloodborne pathogens
- Dispose of needle and holder as a single unit

# Needle-Based Safety

Needle is always bevel up toward the pink safety shield

Same BD PrecisionGlide™ needle quality and sharpness

Safety shield is an "integral" part of the needle

Thumb pad for easy, one-handed activation

Pre-load line

Single Use Only

Engraved text to help drive compliance


# Instructions for Use



Holding both pink shield and green cap, twist and remove white cap.



While holding the needle firmly, screw holder onto needle until it fits securely.

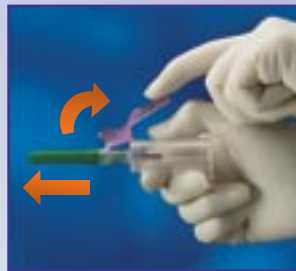

(a) Rotate pink safety shield back toward the holder.  
(b) Twist and pull green needle cap straight off.



Perform venipuncture according to your facility's established procedures.



Immediately after removing needle from vein, cover needle by pushing pink safety shield forward with thumb. An audible click may be heard. Lock into place and inspect. **DO NOT** attempt to engage shield by pressing against hard surface.



**DO NOT** remove needle from holder. Dispose of the needle and holder as one unit into nearest sharps container. **DO NOT REUSE.**