

Special Handling of Specimens Intended for Light-Sensitive Analyte Testing

Transport these specimens
in foil wrapping



Follow your facility's
procedures for specimen
handling.

Examples of light-sensitive analytes:

- Bilirubin
- Erythrocyte
Protoporphyrin
- Carotene

BD Diagnostics
Preanalytical Systems


BD Global Technical Services 1.800.631.0174

BD Customer Service 1.888.237.2762

www.bd.com/vacutainer

Special Handling of Chilled Specimens

Certain analytes must be preserved prior to analysis by keeping the specimen chilled. To ensure accurate results of such specimens, transport them in an ice slurry.

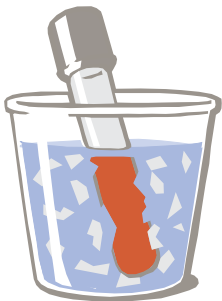

Follow your facility's procedures for specimen handling.

Examples of analytes requiring chilled specimen transport:

- ACTH
- Acetone
- Angiotensin Converting Enzyme (ACE)
- Blood Ammonia
- Catecholamines
- Free Fatty Acids
- Lactic Acid
- Pyruvate
- Renin Activity

BD Diagnostics Preanalytical Systems

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2004 BD. 6/04 VS5732-2
www.bd.com/vacutainer