



## BD Vacutainer™ Safety-Lok™ Blood Collection Set


# Safety Shield Activation

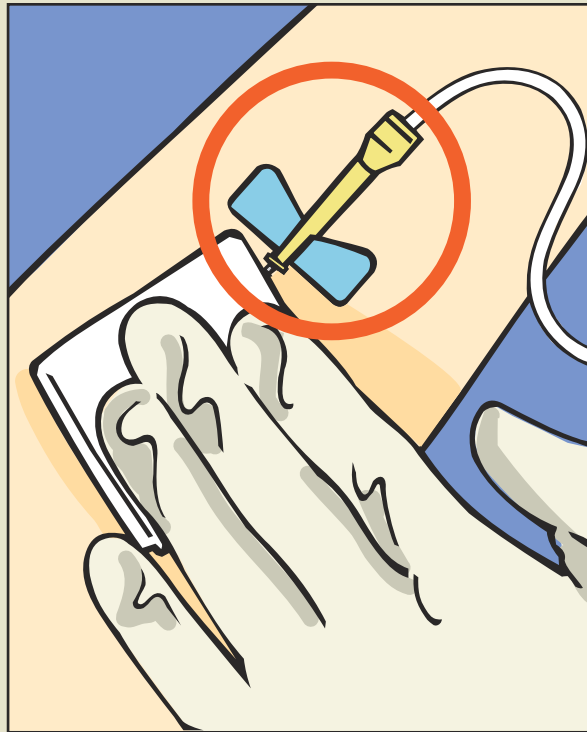


**1** Upon completion of collection, apply light pressure to site using three fingers as shown

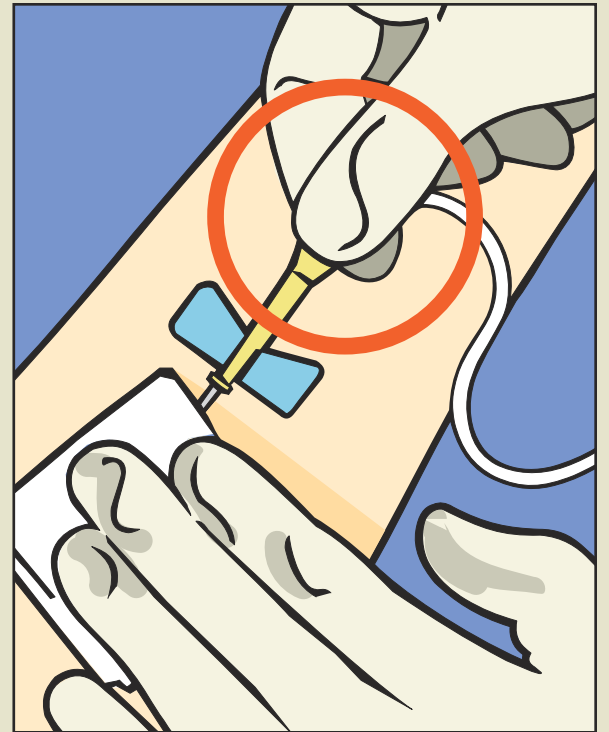

**2** Withdraw blood collection set by grasping the translucent yellow safety shield grip area with the thumb and index finger

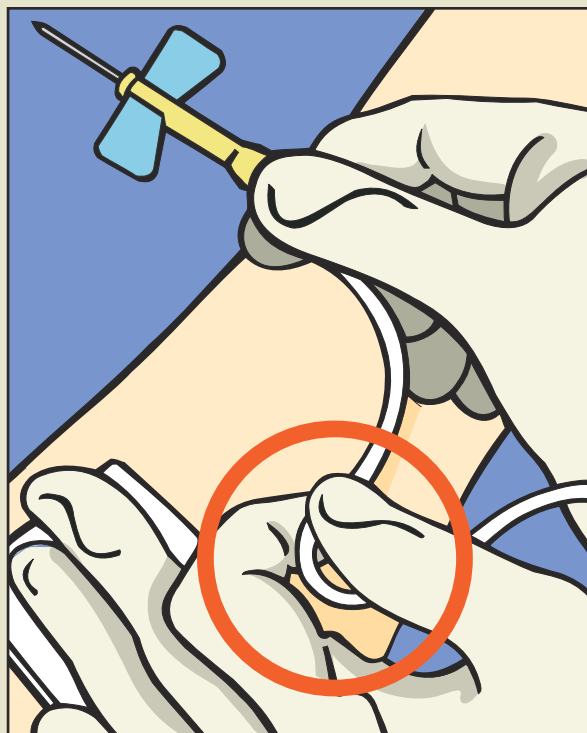

**3** With the opposite hand, grasp tubing between thumb and index finger

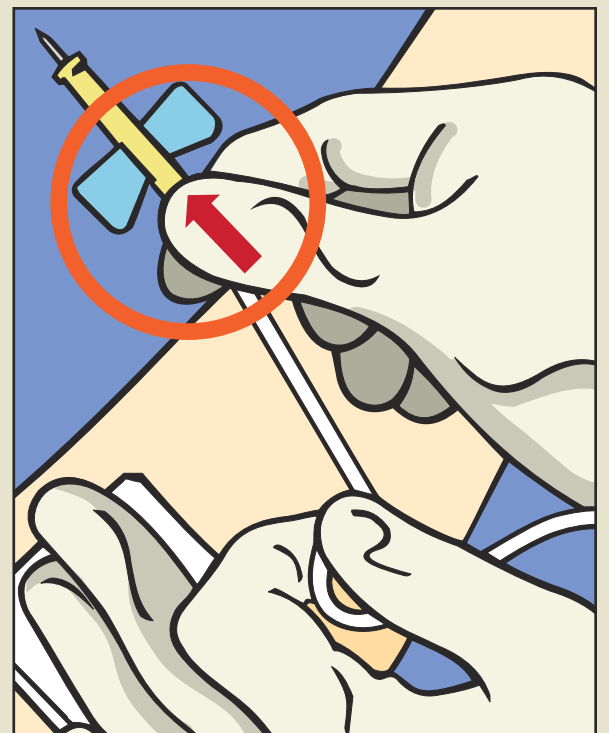

**4** Push the yellow shield forward until the safety shield is locked in place. Discard immediately into an approved sharps container

**BD Vacutainer Technical Services: 1.800.631.0174**  
**BD Customer Service: 1.888.237.2762**  
[www.bd.com/vacutainer](http://www.bd.com/vacutainer)

**SAFETY MESSAGE:**

Handle all biologic samples and blood collection sharps (lancets, needles, luer adapters and blood collection sets) in accordance with the policies and procedures of your facility.

Obtain appropriate medical attention in the event of any exposure to biologic samples (for example, through a puncture injury) since they may transmit viral hepatitis, HIV (AIDS), or other infectious diseases.

Utilize a built-in needle protector, if the blood collection device provides one. BD does not recommend resheilding used needles, but the policies and procedures of your facility may differ and should always be followed.

Discard all blood collection sharps in biohazard containers approved for their disposal. Follow your institution's policy for safe disposal of all medical waste.