

# VACUTAINER® Brand Urine Transport System

**The safe and simple way  
to collect and transport  
a urine sample.**



- VACUTAINER Brand PLUS (plastic) Tube and transfer device enables cleaner transfer of urine sample
- Ensures reliable specimens for accurate results

## Directions for use:



1. Submerge tip of transfer unit in specimen.



2. Push tube into holder all the way. Hold in position until flow stops.



3. Remove tube, leaving holder in specimen container.

Reorder #	Units/Case	Units/Box	Urine Collection and Transport Bulk Packages
364966	1000	100	Integrated sampling device, bulk packed
364979	1000	100	(16mm x 100mm) 10mL draw evacuated round bottom PLUS (plastic) tube for urinalysis, bulk packed

For further information, please read product circular. If you have any questions, please call Technical Services at 1-800-631-0174.

**VACUTAINER  
BRAND  
QUALITY  
MAKES THE  
DIFFERENCE**

**BECTON  
DICKINSON**