**REF 50-7265**

# PleurX™ Lockable Drainage Line Kit

- en** Lockable Drainage Line Kit
- fr** Kit de ligne de drainage verrouillable
- de** Arretierbares Verbindungsschlauchkit
- it** Kit del tubetto di drenaggio bloccabile
- es** Kit de Línea de Drenaje Bloqueable
- pt** Kit da linha de drenagem bloqueável
- nl** Vergrendelbare drainageslangkit
- sv** Kit för låsbar dränageslang
- fi** Lukittava laskuputkisetti
- no** Låsbart drenasjeslangesett

**USA** Rx Only

**STERILE** **EO**

For Single Use Only

© 2012, CareFusion Corporation or one of its subsidiaries. All rights reserved.

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12279870 • 2012-06

Product and packaging do not contain natural rubber latex.

Contains phthalates

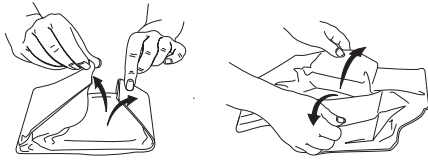
For product inquiries or technical assistance:  
[pleurx-info@carefusion.com](mailto:pleurx-info@carefusion.com)  
[www.carefusion.com/pleurx](http://www.carefusion.com/pleurx)

PleurX, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its subsidiaries.

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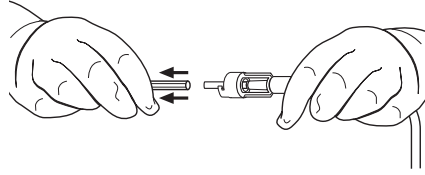
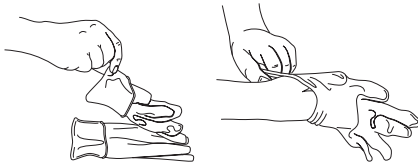
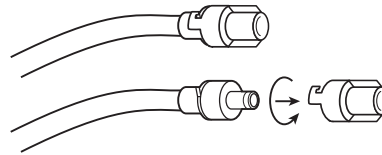
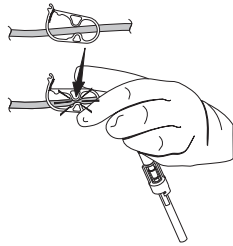
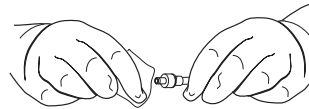
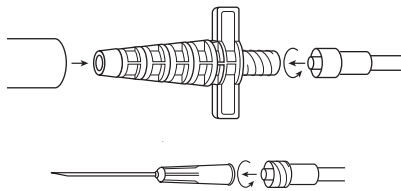
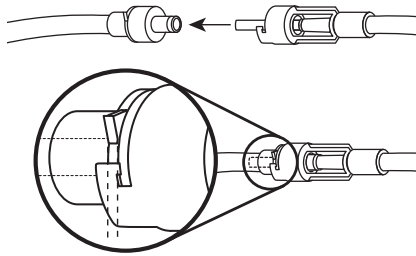
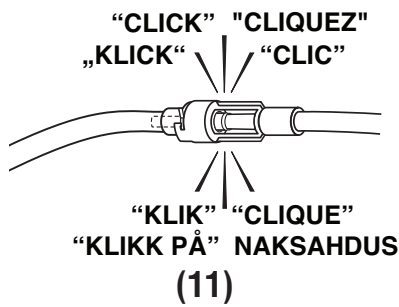
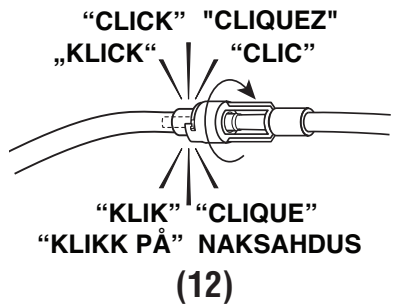
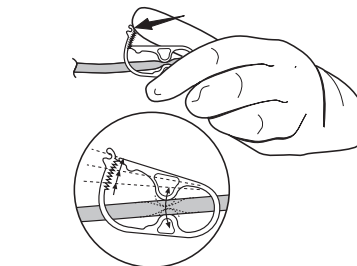
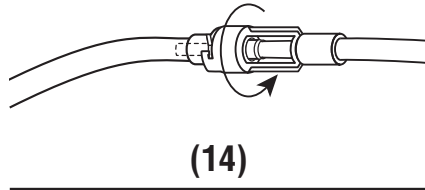
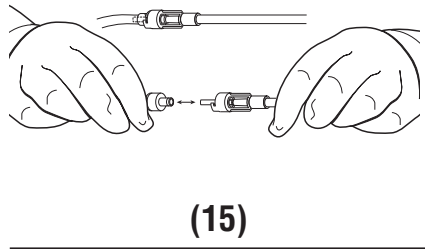
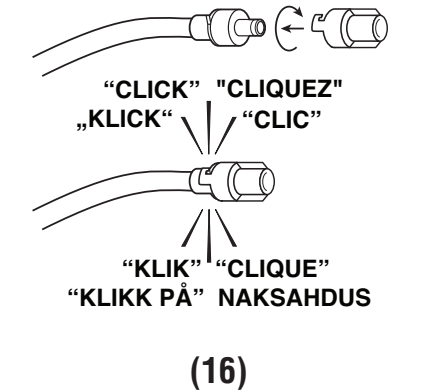
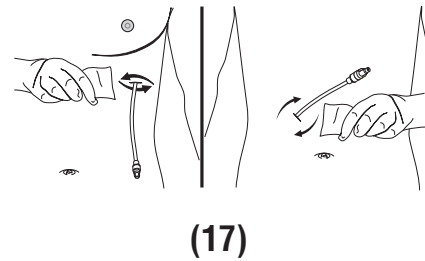
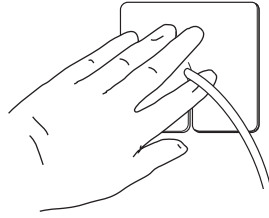
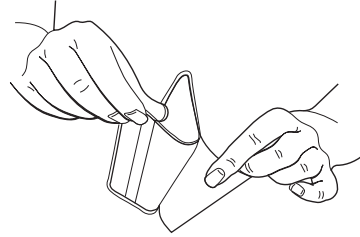
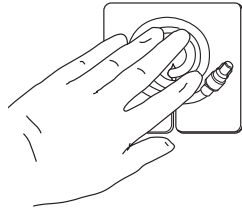
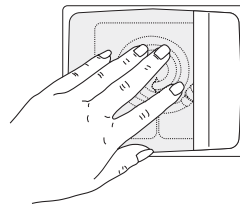
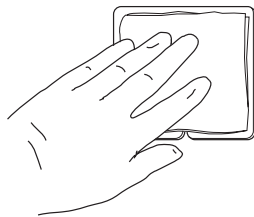
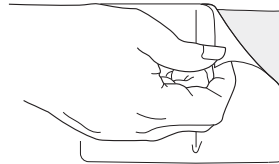
### Kit Contents

CSR wrap containing the following:

- 1 - Drainage Line with Lockable Access Tip
- 1 - 5-in-1 Drainage Line Adapter
- 1 - Valve Cap
- 1 - Blue **Emergency** Clamp
- 4 - Alcohol Pads
- 1 - Pair Gloves
- 1 - Foam Catheter Pad
- 4 - Gauze Pads, 4" x 4" (10.2 cm x 10.2 cm)
- 1 - Self-Adhesive Dressing

**Note:** -60 cm H<sub>2</sub>O = -1.7 in Hg = -44 mm Hg = -.8 psi

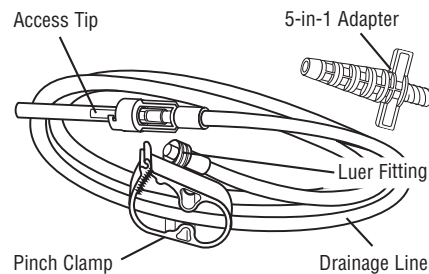
### Product Description

This PleurX Drainage Line Kit is designed to be used exclusively with the PleurX Pleural or Peritoneal Catheter for drainage of the pleural or peritoneal cavity.

### Indications

The Drainage Line Kit is indicated for use only with the PleurX Catheter for intermittent drainage. The Drainage Line Kit is used to drain fluid using standard wall suction, water seal drainage system, vacuum bottle, or other appropriate method (portable suction).

### PleurX Drainage Line

(Figure 1)

### Warnings

Do not drain more than 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen at any one time.

It is normal to feel some discomfort or pain when draining fluid. If discomfort or pain is experienced when draining, clamp the drainage line to slow or stop the flow of fluid for a few minutes. If severe pain or discomfort persists, contact the physician. Pain may be an indication of infection.

When draining with glass vacuum bottles, do not use a needle larger than 17 G. If wall suction or portable suction is used, it must be regulated to -60 cm H<sub>2</sub>O or less.

Make sure that the portable suction unit is capable of regulating to -60 cm H<sub>2</sub>O or less. Check portable suction unit instructions for use prior to draining.

Keep the valve on the PleurX Catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

Do not use scissors or other sharp objects near the PleurX Catheter.

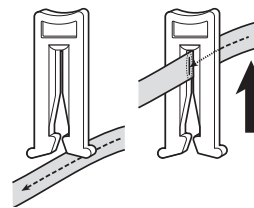
Do not put anything except the access tip of the drainage line into the PleurX Catheter valve since this could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

Precautions should be taken to ensure the drainage line is not tugged or pulled.

The patient should not be left unattended during drainage.

### If you accidentally cut the catheter or damage the valve, follow these steps:

- a. Pinch the catheter closed between your fingers.
- b. Slip the blue **emergency** slide clamp over the catheter and push the catheter completely into the small end of the clamp. This will close the catheter. (Figure 2)
- c. Notify the doctor immediately.

(Figure 2)

### Cautions Rx Only

For single use only. Re-use may result in a nonfunctional product or contribute to cross contamination.

When connecting to a glass vacuum bottle, make sure the pinch clamp on the drainage line is fully closed. Otherwise, it is possible for some or all of the vacuum in the bottle to be lost.

The pinch clamp must be fully closed to occlude the drainage line. When not connected to a suction source, make sure the pinch clamp is fully closed otherwise the drainage line may allow air into the body or let fluid leak out.

Make sure that the valve and the access tip are fully connected when draining. If they are accidentally separated, they may become contaminated. If this occurs, clean the valve with an alcohol pad and use a new drainage set to avoid potential contamination.

The alcohol pads are flammable. Do not expose the pads to an open flame.

Contains Phthalates. The benefit of treatment outweighs the remote possibility of exposure to phthalates.

**Chest:** Potential complications of draining the pleural space include, but may not be limited to, pneumothorax, re-expansion pulmonary edema, hypotension, circulatory collapse and infection.

**Abdomen:** Potential complications of draining fluid from the abdomen include, but may not be limited to, hypotension, circulatory collapse, electrolyte imbalance, protein depletion, ascites leakage, peritonitis, wound infection, and loculations of the peritoneal space.

If fluid spills, be sure to clean. Use soap and warm water for clothing and skin. Use bleach for household items.

**Contact the patient's physician if:**

- The patient develops fever (body temperature above 100.5° F (38°C), or you notice any redness, swelling, oozing or pain at the exit site. These may be signs of infection that may require treatment.
- Shortness of breath is not relieved after draining 1000 ml from the chest or 2000 ml from the abdomen at one time.
- The patient continues to experience symptoms, but little or no fluid drains from the catheter.
- Less than 50 ml drainage in three (3) drainage procedures in a row.
- The appearance (color, thickness, etc.) of the fluid changes significantly between drainages.

**Sterility**

This product has been sterilized and is for single use only. Do not re-sterilize. CareFusion will not be responsible for any product that is re-sterilized, nor accept for credit or exchange any product that has been opened but not used.

Do not use if package is damaged.

**General Guidelines**

1. These instructions are provided as a guide to the clinician to perform a drainage procedure and are intended to supplement any hospital developed protocols, clinical drainage protocols or appropriate drainage device instructions for vacuum sources other than the PleurX bottle. Ensure that you are familiar with these instructions prior to beginning the procedure.
2. When using the Drainage Line, it may be used to drain at a suction level of -60 cm H<sub>2</sub>O or less for as long as it takes to drain 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen per day. The volume of fluid removed should be based on the individual patient's status and the risks for over draining, including re-expansion pulmonary edema in the chest or hypotension for ascites patients.

**How to Drain Using the Drainage Line Kit**

The drainage and dressing procedures should be performed using aseptic technique. The patient should be drained as directed by their doctor, usually every one to two days. Do not change frequency or drain more fluid than their doctor has

recommended without first consulting their doctor. Make sure you are familiar with the suction method prior to draining.

**Warning:** Do not use scissors or other sharp objects near the PleurX Catheter.

**Note:** The blue **emergency** slide clamp should be used if you accidentally cut the catheter or damage the valve. See **Warnings. (Figure 2)**

**Prepare to Drain**

**Warning:** Keep the valve on the PleurX Catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

**Note:** Dispose of all used product in accordance with applicable local, state, and federal regulations. Used product may present a potential biohazard.

1. Set up a clean, clear workspace on a table or counter.
2. Thoroughly wash your hands with soap and water for at least 1 minute.
3. Remove the dressing from over the catheter. Be sure not to tug on the catheter.

**Caution:** If the patient develops a fever (body temperature above 100.5°F (38°C), or you see any redness, swelling or fluid around the catheter, finish this drainage procedure and contact the patient's doctor or nurse.

4. Thoroughly wash your hands again with soap and water for at least 1 minute.
5. Open the Drainage Line Kit.
6. Set bundle with blue wrapping on your workspace with the flap side up. Carefully unfold the blue wrapping by pulling on the outside of the wrapping. Leave the enclosed items on the wrapping. The items and the inside of the wrapping are sterile. Do not touch them with your ungloved hands or other non-sterile items. **(Figure 3)**
7. Both gloves fit either hand. Be careful not to let the outside of the gloves touch anything non-sterile, such as your skin or clothing. Put on gloves as shown. **(Figure 4)**
8. Tear open the three alcohol pads, but do not remove the pads from their pouches. Place them on the blue wrapping.

**Connect the Drainage Line**

**Warning:** The patient should not be left unattended during drainage.

**Warning:** Do not put anything except the access tip of the drainage line into the PleurX Catheter valve since this could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

**Warning:** Precautions should be taken to ensure the drainage line is not tugged or pulled.

**Warning:** When draining with glass vacuum bottles, do not use a needle larger than 17 G. If wall suction or portable suction is used it must be regulated to -60 cm H<sub>2</sub>O or less.

Proofed by: _____	Date: _____
Dimensions checked: _____	Copy checked: _____

**Warning:** Some portable suction units are not capable of regulating to -60 cm H<sub>2</sub>O or less. Read the portable suction unit instructions for use prior to draining.

1. Clamp the drainage line completely closed using the pinch clamp found on the tubing. **(Figure 5)**
2. If using wall suction or portable suction, attach the 5-in-1 adapter to the Luer fitting on the drainage line. If using a vacuum bottle, attach a 17 G needle to the Luer fitting on the drainage line. **(Figure 6)**
3. Connect the 5-in-1 adapter to the vacuum/suction source.
4. Hold the drainage line near the access tip and remove the protective cover by twisting it and pulling gently. Take care to avoid contaminating the access tip. **(Figure 7)**
5. Hold the base of the catheter valve and remove the cap by twisting it counterclockwise and pulling gently. Discard the cap. Take care to avoid contaminating the valve. **(Figure 8)**
6. While holding the base of the catheter valve in one hand, clean around the valve opening with a new alcohol pad. **(Figure 9)**
7. Continue holding the catheter near the valve. Carefully align the access tip and valve as shown in **(Figure 10)** and insert the access tip into the catheter valve and advance it completely into the valve. You will feel and hear a click when the access tip and valve are securely connected. **(Figure 11)**
8. If desired, lock the access tip to the catheter valve by twisting the access tip until you feel and hear a second click. **(Figure 12)**

#### Drain Fluid

**Warning:** It is normal to feel some discomfort or pain when draining fluid. If discomfort or pain is experienced when draining, clamp the drainage line to slow or stop the flow of fluid for a few minutes. If severe pain or discomfort persists, contact the physician. Pain may be an indication of infection.

**Warning:** Do not drain more than 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen at any one time.

1. Release the pinch clamp on the drainage line to begin drainage. You can reduce the flow rate by squeezing the clamp partially closed. **(Figure 13)**
2. If you need to change to a new bottle/canister/water seal device for any reason, squeeze the pinch clamp on the drainage line completely closed. Remove the drainage line from the vacuum/suction source and connect to a new bottle/canister/water seal device. Release the pinch clamp to resume draining.
3. When flow stops or the desired amount of fluid has been removed, squeeze the pinch clamp on the drainage line completely closed. **(Figure 5)**
4. If locked, twist the access tip to unlock it from the catheter valve. **(Figure 14)**

5. With the drainage line in one hand and the catheter valve in the other hand, pull the access tip out of the valve in a firm, smooth motion. **(Figure 15)**
6. Clean the catheter valve with a new alcohol pad.
7. Place the new cap over the catheter valve and twist it clockwise until it snaps into its locked position. **(Figure 16)**
8. If required, record the drainage volume.
9. Disconnect the vacuum/suction source and dispose of the used drainage line and bottle/canister/water seal device in accordance with applicable local, state, and federal regulations.

**Caution:** If fluid spills, be sure to clean. Use soap and warm water for clothing and skin and use bleach for household items.

#### Dress the Catheter

1. Clean around the catheter exit site with a new alcohol pad. **(Figure 17)**
- Note:** Make sure the exit site and skin around the catheter are dry before completing the dressing procedure.
2. Place the foam catheter pad around the catheter. **(Figure 18)**
  3. Wind the catheter into loops and place it over the foam pad. **(Figure 19)**
  4. Cover the catheter with up to four gauze pads. **(Figure 20)**
  5. Remove gloves from both hands.
  6. The self-adhesive dressing has three (3) layers:
    - a. Clear shiny plastic covering
    - b. Clear wound dressing
    - c. Two piece white paper backing
  7. Take the self-adhesive dressing and peel away the larger of the two white pieces of paper backing from the dressing. **(Figure 21)**
  8. Center the dressing over the gauze pads and press it down. **(Figure 22)**
  9. There is a clear shiny plastic covering that needs to be removed from the dressing. Starting at one of the corners where the white paper backing remains, bend the shiny plastic covering back slightly and pinch the corner of the white paper and the clear wound dressing to separate the shiny plastic covering from the clear wound dressing.

**Note:** It may be easier to separate a corner of the white paper from the clear wound dressing and use the tip of your finger or fingernail to start separating the shiny plastic covering from the dressing. (This is similar to the way you would remove a piece of tape or an adhesive tag.) Peel the shiny plastic covering from the dressing. **(Figure 23)**

10. Remove the remaining white paper backing from the clear wound dressing and press it down.

**Note:** Product and packaging do not contain natural rubber latex.

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