**REF 50-7290**

# PleurX™ Procedure Pack

- en Procedure Pack
- fr Trousse pour procédure de drainage
- de Verbandmaterial-Set
- it Procedure Pack
- es Paquete de Procedimiento
- pt Conjunto de procedimento
- nl Procedurepakket
- sv Procedurpaket
- fi Toimenpidepakkaus
- no Prosedyrepakke

**USA** Rx Only

**STERILE** **EO**

For Single Use Only

Product and packaging do not contain natural rubber latex.

Contains phthalates

© 2012, CareFusion Corporation or one of its subsidiaries. All rights reserved.

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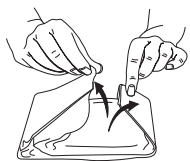
PleurX, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its subsidiaries.

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
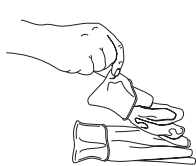
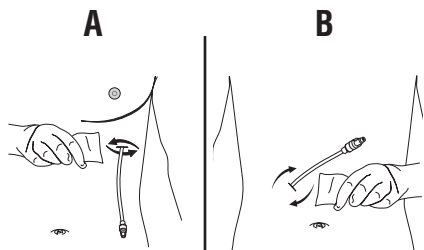
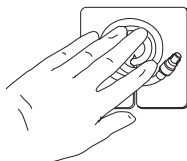
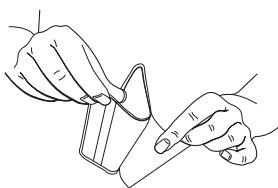
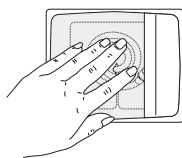
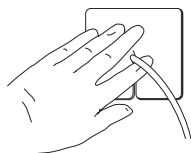
Self-Adhesive Dressing

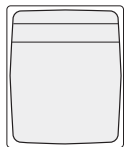(A)

(B) Blue Wrapping around the following:

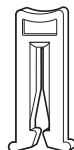
Alcohol Pads

(C)

Blue **Emergency** Slide Clamp

(F)


Gloves

(D)


Gauze Pads

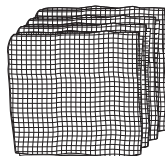(G)


Catheter Valve Cap

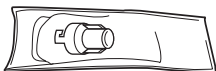(E)

Foam Catheter Pad

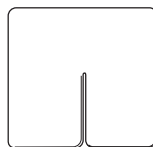(H)

## Instructions for Use

### Kit Contents

Self-Adhesive Dressing (A)

Blue Wrapping (B)

(Wrapped around the following Items):

3 Alcohol Pads (C)

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1 Valve Cap (E)

1 Blue **Emergency** Slide Clamp (F)

4 Gauze Pads 4" x 4" (10 cm x 10 cm) (G)

1 Foam Catheter Pad (H)

### Product Description

The PleurX™ Procedure Pack contains the necessary supplies for dressing of the PleurX Pleural or Peritoneal catheter after performing a drainage procedure.

You should have been trained by a doctor or nurse in the correct way to use the items in this PleurX Procedure Pack. If you have any problems or questions about dressing the catheter and exit site, contact your doctor or nurse. Read this entire pamphlet carefully before dressing the catheter and exit site.

### Indication

The PleurX Procedure Pack is indicated for dressing of the PleurX Pleural or Peritoneal Catheter and exit site.

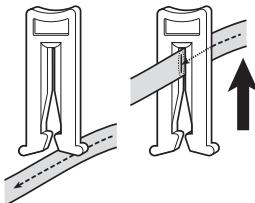
### Warnings

Keep the valve on the PleurX catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

Do not use scissors or other sharp objects near the PleurX catheter.

**If you accidentally cut the catheter or damage the valve, follow these steps:**

- Pinch the catheter closed between your fingers.
- Slip the blue **emergency** slide clamp over the catheter and push the catheter completely into the small end of the clamp. This will close the catheter. **(Figure 1)**
- Notify your doctor immediately.

(Figure 1)

### Cautions

For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

If fluid spills, be sure to clean. Use soap and warm water for clothing or skin and use bleach for household items.

The alcohol pads are flammable. Do not expose the pads to an open flame.

If you develop a fever (body temperature above 100.5 °F [38°C]), see any redness, swelling, or fluid around the catheter, finish this drainage procedure and contact your doctor or nurse.

Contains Phthalates. The benefit of treatment outweighs the remote possibility of exposure to phthalates.

### Sterility

The PleurX Procedure Pack has been sterilized. This product is for single use only. Do not re-sterilize.

Do not use if package is damaged.

CareFusion will not be responsible for any product that is resterilized, nor accept for credit or exchange any product that has been opened but not used.

### How to use the PleurX Procedure Pack

**If you are performing a drainage procedure refer to the PleurX Drainage Procedure Instructions for Use.**

**If you are not performing a drainage procedure at this time and only dressing the catheter exit site:**

**Note:** The valve cap will not be used.

- Set up a clean clear workspace on a table or counter.
- Thoroughly wash your hands with soap and water for at least 1 minute.
- Remove the dressing from over the catheter. Be sure not to tug on the catheter.
- Thoroughly wash your hands again with soap and water for at least 1 minute.
- Open the PleurX Procedure Pack. **(Figure 2)**
- Set the bundle with blue wrapping on your workspace with the flap side up. Carefully unfold the blue wrapping by pulling on the outside of the wrapping. Leave the enclosed items on the wrapping. The items and the inside of the wrapping are sterile. Do not touch them with your ungloved hands or other non-sterile items.
- Put the gloves on. Both gloves fit either hand. Be careful not to let the outside of the gloves touch anything non-sterile, such as your skin or clothing. Put on gloves as shown. **(Figure 3)**
- Clean around the catheter site with a new alcohol pad. **(Figure 4A and Figure 4B)**

**Note:** Make sure the exit site and skin around the catheter are dry before completing the dressing procedure.

9. Place the foam catheter pad around the catheter. **(Figure 5)**
10. Wind the catheter into loops and place it over the foam pad. **(Figure 6)**
11. Cover the catheter with up to four gauze pads. **(Figure 7)**
12. Remove gloves from both hands.
13. The self-adhesive dressing has three (3) layers:
  - a. Clear shiny plastic covering
  - b. Clear wound dressing
  - c. Two piece white paper backing
14. Take the self-adhesive dressing and peel away the larger of the two white pieces of paper backing from the dressing. **(Figure 8)**
15. Center the dressing over the gauze pads and press it down. **(Figure 9)**
16. There is a clear shiny plastic covering that needs to be removed from the dressing. Starting at one of the corners where the white paper backing remains, bend the shiny plastic covering back slightly and pinch the corner of the white paper and the clear wound dressing to separate the shiny plastic covering from the clear wound dressing.  
**Note:** It may be easier to separate a corner of the white paper from the clear wound dressing and use the tip of your finger or fingernail to start separating the shiny plastic covering from the dressing. (This is similar to the way you would remove a piece of tape or an adhesive tag.) Peel the shiny plastic covering from the dressing. **(Figure 10)**
17. Remove the remaining white paper backing from the clear wound dressing and press it down.  
**Note:** Product and packaging are not made with natural rubber latex.

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