


BD Eclipse™ Needle

Important usage guidelines

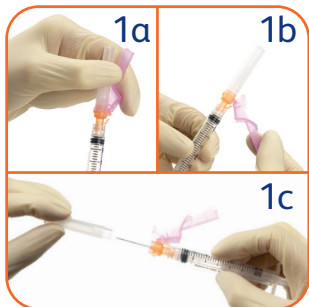

1. Attachment

1a. Attach the BD Eclipse™ Needle to a BD Luer-Lok™ syringe.

1b. Pull back the safety cover.

1c. Remove the clear needle shield.

Note: Bevel up=Safety cover up




2. Aspiration

Draw up medication in accordance with your institution's established protocol.



3. Injection

Administer medication in accordance with your institution's established protocol.

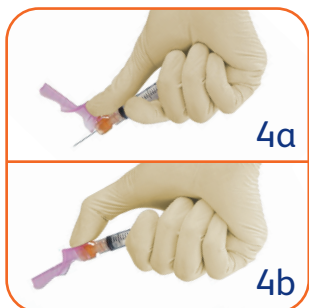

4. Activation

Activate safety mechanism immediately after injection.

4a. Center your thumb or forefinger on the textured finger pad.

4b. Push the safety cover forward over the needle until you hear or feel it lock.

Visually confirm that the needle is covered.

If unable to activate, discard immediately in an approved sharps container.

Use one handed technique and activate away from self and others. For greatest safety, ONLY use the wide textured finger pad area to activate safety cover.

Discard after single use in an approved sharps container in accordance with applicable regulations and institutional policy.

Remember

- The BD Eclipse™ Needle is not to be used with Luer-Slip syringes as the needle may become disengaged from the syringe when activating the safety cover.
- Follow your institution's established protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.
- Activation of the protective mechanism may cause minimal splatter of any fluid that is remaining in the needle after injection.

This guide is not intended as a replacement for the detailed information contained in the IFU. Please consult BD product labels and package inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

1 Becton Drive, Franklin Lakes, NJ, 07417, USA

Scan this code to watch a video on how to use the BD Eclipse™ Needle



[bd.com](https://www.bd.com)



BD, the BD Logo, Eclipse, and Luer-Lok are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2023 BD. All rights reserved. BD-14230 (05/23)