English

Temperature Simulator Keys

Instructions for Use

Warning: Medivance supplies temperature simulators (fixed value resistors) for testing, training and demonstration purposes only. Never use this device, or other method, to circumvent the normal patient temperature feedback control when the system is connected to the patient. Doing so exposes the patient to the hazards associated with severe hypo- or hyper-thermia.

The intended use for the temperature simulator keys is to test, train and demonstrate the Arctic Sun™ Temperature Management System. The Temperature Simulator Keys are Not for Clinical Use. Each key has a fixed temperature value that represents an artificially simulated patient temperature (within ±0.2 °C) to the Arctic Sun™ Temperature Management System.

Visually inspect the Temperature Simulator Keys prior to use. There are three configurations that the Temperature Simulator Keys can be ordered. Three keys attached on a Simulator Key Hang Tag (748-00), a 37°C key packaged alone (777-00), or within an In-Service Training Kit (771-00).

The three following keys should be present on the Simulator Hang Tag:

- 39°C - represented in the color Red
- 37°C - represented in the color White
- 33°C - represented in the color Blue

To simulate a patient temperature, select the appropriate Temperature Simulator Key for the desired testing, training, or demonstrating outcomes. Connect the selected Temperature Simulator Key into the patient temperature cable. Once connected, insert the opposite end of the patient temperature cable into the patient temperature port on the back side of the Arctic Sun™ Temperature Management System.

Temperature Simulator Keys are Not for Clinical Use and should never be used while a patient is undergoing targeted temperature therapy. In order for the patient to receive proper targeted temperature management, a patient temperature probe must be connected to the patient temperature cable which is connected to the Arctic Sun™ Temperature Management System.

Prior to targeted temperature therapy, the clinician should always:

- Check that there is not a temperature simulator key connected to the patient temperature cable.
- Verify that the patient temperature probe is properly placed on the patient.
- Verify that the patient temperature probe is connected to the Arctic Sun™ Temperature Management System.

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BD, the BD Logo, Arctic Sun and Medivance are trademarks of Becton, Dickinson and Company or its affiliates. © 2024 BD. All rights reserved.